

BIOPSY BAGS



CODE	DIMENSIONS	PACKAGING
07-00005	30 x 45 mm	1000 pz
07-00004	45 x 60 mm	1000 pz
07-00003	73 x 75 mm	1000 pz



Dispositivo medico – diagnostico in vitro



Manufacturer: Bio-Optica Milano S.p.A.



Disposable product

Date of emission: 23/03/2020

Rev. 03

Small nylon bags for the processing of biopsy and histological specimens.

INSTRUCTIONS


1. Empty the fixative and the specimen in the bag, the bag filter the fixative and the biopsy remain perfectly inside. Close the bag and put it in a standard biopsy cassette. Proceed as a usual processing.



2. After processing extract the bag from the cassette, open it with care extending the edges.



3. Extract the specimen and embed it in paraffin.

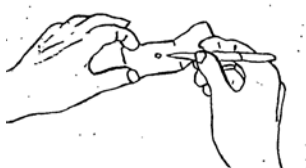

TECHNICAL SPECIFICATIONS

Bio-Optica biopsy bags are produced by cutting and ultrasound solder; the thin mesh avoids the little sample's lost.

Technical data	Composition	Nylon
	Pore size	about 0,184 mm
	Dimensions	See tabelle page 1
Packaging	Primary container	Plastica folder
	Secondary container	Cardboard box
Storage	Storing conditions	None
	Validity	Not applicable to this kind of product
Warning and precautions	Classification of the products	For specialized technical operators. Product not classified as dangerous.
	Disposal	Dispose in conformity with local laws.

Date of emission: 23/03/2020

Rev. 03